


CLAVE* Needle-free Connector

USAGE GUIDELINES

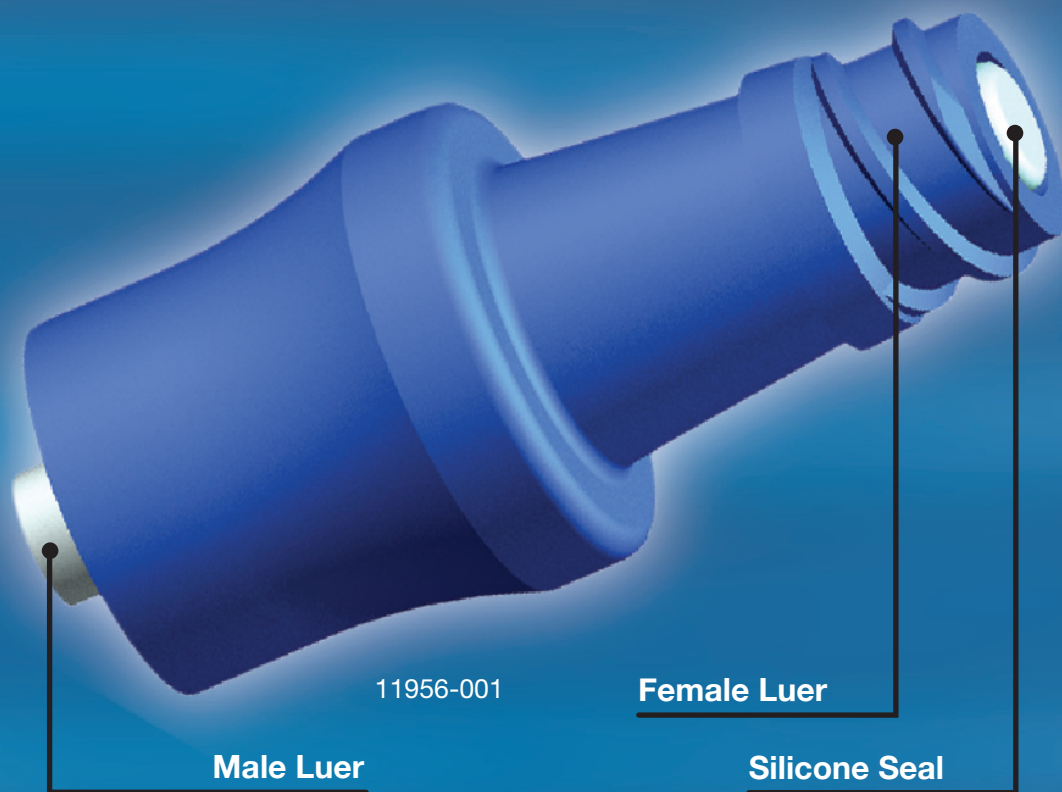

For Needle-free Access to Central, Peripheral and Arterial Lines

1

Prime the CLAVE

1. Remove from package and attach IV administration set or priming syringe to **female luer** of CLAVE.
2. Prime CLAVE per facility protocol, taking care to invert device to expel air.
3. Remove protective cap and attach **male luer** of CLAVE to IV extension set or catheter.



2

Swab and Access the CLAVE

1. Swab **silicone seal** of CLAVE Connector before each access with desired disinfectant per facility protocol.
2. To access with luer lock or luer slip IV device, grasp CLAVE and firmly push and twist device into CLAVE until tight. To remove device, grasp CLAVE and twist apart until loose.

NOTE: If using a spin lock device, first push the male tip into the CLAVE by inserting it into the silicone seal and rotate 1/4 turn to secure the connection. Then "lock down" the spin collar. This will ensure a proper connection and optimal flow rates.

3. The CLAVE Connector is a closed barrier to bacteria.
Do not use end caps.

CLAVE Features:

- Eliminates sharps entirely from IV therapy
- Flow rate average = 185 cc/min @ 36" head height
- Backpressure = 60 psig
- Lipid, blood, chemo-drug compatible
- Residual volume (dead space) = 0.06 cc
- Contains no natural rubber latex

3

Flush the CLAVE

1. Flush the CLAVE Connector after each use per facility protocol.

CAUTION: It is recommended this device be changed per validated facility protocol.

